



Protocol of use for finisher Gentlefile-Brush

Purpose:


Activation of disinfection and cleaning solutions on the dentinal walls of the canals; and to reach the irregular spaces in the canals.

Function:

At high RPMs, the strands of the brush open to the full variable diameter of the canal. This cleans remaining debris in the canal and brings the irrigants into intimate contact with the dentinal surface.

Protocol:

1. Complete the filing of the canal, such that the canal is ready to be filled.
2. Flood the canals with irrigant of choice.
3. Insert the Gentlefile-Brush into the handpiece.
4. Insert the Gentlefile-Brush into the canal with the handpiece, to near the apex. Activate the handpiece and very slowly withdraw the Gentlefile-Brush.
5. This should take around 10-20 seconds.
6. Dry the canal and fill.

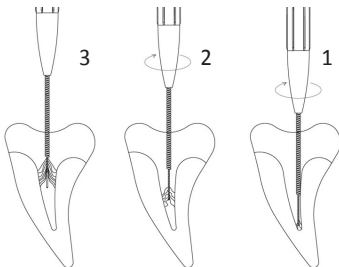

Note: Don't stop the withdrawal movement backward in the canal. Don't try to push into the canal while operating GF-Brush. If the GF-Brush stays in one place, a ledge can develop.

MEDICNRG™
Precise Endo Technology
www.medicnrg.com



MedicNRG Ltd.
Kibbutz Afikim 15148 ISRAEL
Tel: +972-4-675-4217
Fax: +972-4-675-4278
Email: info@medicnrg.com
Web: www.medicnrg.com



Obelis s.a.
Bd Général Wahis 53 1030
Brussels, Belgium
Tel: +(32) 2 732-59-54
Fax: +(32) 2 732-60-03
E-mail: mail@obelis.net

MPRB-GFB1-00